**BIOPTRON®**   
HYPERLIGHT THERAPY SYSTEM By Zepter Group

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ENGLISH (EN)

**CONTENTS**

IMPORTANT SAFEGUARDS	4
RED FILTER	8
ORANGE FILTER	9
YELLOW FILTER	10
GREEN FILTER	11
BLUE FILTER	12
INDIGO FILTER	13
VIOLET FILTER	14
WHITE FILTER	15
TECHNICAL DATA	16
USE AND MAINTENANCE	20
WARRANTY / WARNINGS	22

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## IMPORTANT SAFEGUARDS

### RISK GROUP II ACCORDING TO IEC 62471 (RELATED TO THE DEVICE)

#### FOR THE TREATMENT OF NEWBORNS USE BLINDFOLD (PAG-500-BF).

Treatments of children and people with cognitive impairment should always be performed under adult supervision. Children should be supervised to ensure that they do not play with the appliance. Treatments of children under six years of age and newborns must be performed under supervision of a physician.

Biopton treatments during pregnancy should be followed by a physician.

Consult your doctor before using Biopton on, or in proximity to, cancerous lesions.

The filters should be installed and used only with the Biopton devices recommended by the manufacturer.

#### CONTRAINDICATIONS:

Before using BIOPTRON Hyperlight Therapy System, consult your physician, especially in the event of:

- Photo-allergy • Porphyria (all types) • Actinic reticulosis (photo-allergic contact dermatitis) • Cheilitis exfoliativa actinica (Inflammation of the lower lip due to radiation) • Erythema exsudativum multiforme • Hydroa vacciniformis • Xeroderma pigmentosum (extreme sensitivity to UV rays) • Retinal diseases • Taking of photosensitizing pharmaceuticals/herbs.

**POTENTIAL CONTRAINDICATIONS:** • Lupus erythematosus • Eye inflammation

Treatments with Biopton Hyperlight Therapy System can be done once or twice daily for as long as needed, if not stated otherwise.

During the sessions, it is possible to use several monochromatic filters in turn.

**BEFORE USE, CONSULT YOUR DOCTOR TO SEE IF THIS THERAPY IS SUITABLE FOR THE INTENDED TREATMENT PURPOSE.**

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## RED FILTER

PAIN RELIEF, PHYSIOTHERAPY,  
ANTI-INFLAMMATORY

### MEDICAL INDICATIONS

#### CHRONIC PAIN (Joint pain):

- recommended treatment time  
4 - 10 min, distance 10 cm
- Rheumatoid arthritis

#### PHYSIOTHERAPY:

- recommended treatment time  
4 - 10 min, distance of 10 cm
- Low back pain
  - Shoulder and neck pain
  - Carpal tunnel syndrome

#### SPORTS MEDICINE:

- recommended treatment time  
4 - 10 min, distance of 10 cm
- Tendinopathies
  - Acute ankle sprains

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
## TECHNICAL DATA

Illustrative spectral graphic image of representative samples (the effective spectrum of a filter can slightly vary from the displayed spectra)

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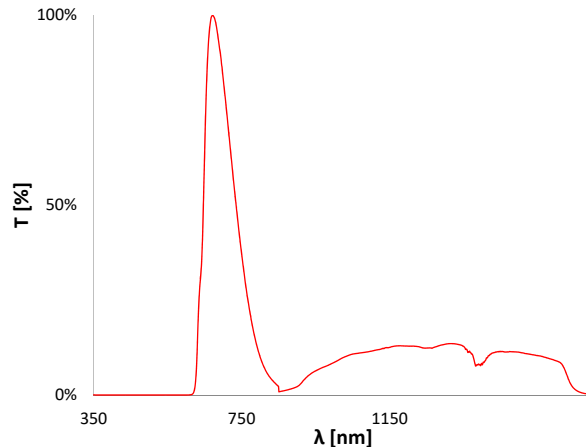**ORANGE FILTER**  
WOUND HEALING**MEDICAL INDICATIONS****WOUND HEALING:**

recommended treatment time  
2 - 8 min, distance of 10 cm

Improve the healing of scars and range of motion  
after burn injury.

- Hypertrophic burn scars
- Postburn scars
- Wounds completely closed.

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
**TECHNICAL DATA**

Illustrative spectral graphic image of representative  
samples (the effective spectrum of a filter can  
slightly vary from the displayed spectra)

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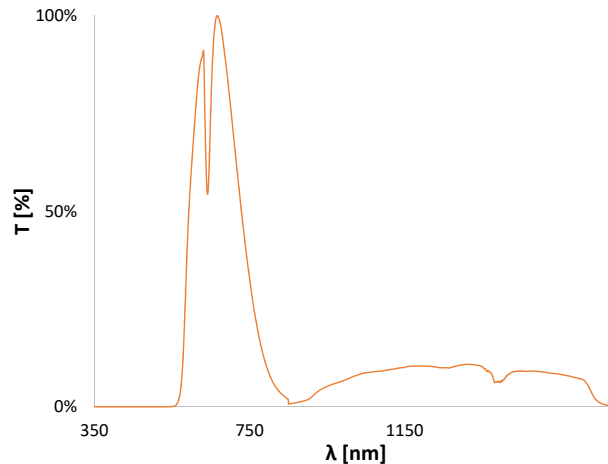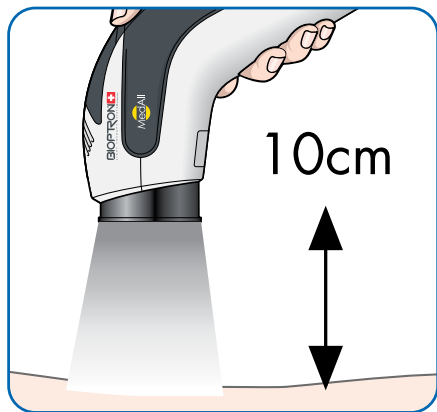
ENGLISH

**HOW TO USE?**

From a distance of 10 cm, position the BIOPTRON device with the desired filter to cover the area you wish (Picture 1) to treat. When treating areas larger than the color filter diameter, treat them one by one according to the recommended time.

**HOW TO CHANGE THE FILTER?**

Carefully and without force take out the filter (Picture 2) to be replaced and insert the desired color filter as indicated in Picture 3. Hold the outer ring of the filter rather than the lens.

**HOW TO CLEAN THE FILTER? (Picture 4)**

1. Remove the filter from the device by holding the outer ring of the filter.
2. Use a soft, wet, lint-free cloth to avoid scratching the glass surface.
3. Dry the filter with another lint-free cloth.

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ENGLISH

**WARRANTY:**

- 5 years if used as intended and if warnings are respected
- This warranty covers material and manufacturing defects
- This warranty does not cover aesthetic changes resulting from ordinary wear and tear that do not affect product performance, or damages caused by product misuse.

**WARNINGS**

1. Do not use any chemicals (e.g. isopropyl alcohol) to clean the inner or outer side of the glass surface and filter ring.
2. Store the filter in a protective pouch to prevent the accumulation of dust and discoloration due to exposure to sunlight.
3. The filter is made of fragile glass, so it must be handled with care!
4. User is not allowed to remove filter lens.

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**BIOPTRON AG**